

Expansion/Distalization & Metals

Models/Scans: Upper and lower model in Class I or Class II stone (models). Complete anterior and posterior anatomy of the arch on which the appliance will be fabricated. Spacers can be placed after the impression/scan is taken.

***Opposing Models:** It is strongly suggested that the opposing arch be sent as well when appliances involving acrylic covering the occlusal surface are requested (ex: bonded expanders) or when the occlusion should be evaluated to prevent interferences (ex: habit crib).

Wax Bites: Only necessary if centric occlusion is not obvious when articulating models.

Notes: Specialty can supply and fit crowns, bands and the new ROCs (removed occlusal crowns) indirectly when anchorage teeth are fully erupted.

***Please evaluate the eruption of teeth selected for anchorage. If anchorage teeth are not fully erupted, please do either of the following: (1) Fit bands/crowns and send loose in a bag (or in impression) for technicians to fit directly or (2) consider other teeth for anchorage.**

