



# Instructions for Use

OnBRACE®










## **Specialty Appliances™ Recommended Bonding Instructions:**

1. Pumice and rinse teeth, dry. Apply 20% polyacrylic acid for 10 seconds to the buccal and lingual surfaces, rinse 10 seconds. Alternatively use 37% phosphoric acid for 30 seconds.
2. Dry and apply Reliawax® interproximally and thin layer occlusally, shine curing light. Prime interior of OnBRACE® with Assure Plus and light cure.
3. Load OnBRACE® half-full with multi-cure glass ionomer cement. Air dry teeth but DO NOT DESICCATE. Seat OnBRACE® Appliance, remove bulk excess with cotton roll, wipe periphery with a primed microbrush. Recommended Adhesive: Reliance GlassLok Liquid (SKU: GlassLok Liquid)
4. Shine curing light 5 seconds mesial, distal, and occlusal, use scaler to remove excess cement, remove interproximal Reliawax.
5. Allow 10 minutes setting time for cement prior to activating OnBRACE® appliance.

## **Specialty Appliances™ Recommended Debonding Instructions:**

1. Using a debonding plier, grasp the buccal attachment and roll the OnBRACE® to the lingual while providing counter-pressure with finger on the opposite side of tooth. Recommended Plier: Prodent Angled Debonding Plier (SKU: 30-542B)
2. For OnBRACE® without buccal attachment: Using a ligature cutter, grasp the buccogingival margin of the OnBRACE® and Peel using a rolling motion to the lingual while providing counter-pressure.
3. Dessicate cement, remove with adhesive removing plier. Remove excess cement with a scaler and a handpiece.

## General symbols

	Catalog number
	Lot number
	Manufacturer
	Date of manufacturer
	Non-sterile
Rx only	Rx only
	Keep dry
	Consult Instructions for Use
	Unique Device Identifier
	Quantity



800.522.4636 | [specialtyappliances.com](http://specialtyappliances.com)